Parallel Bars

Bedieningshandleiding
Operating Instructions
Gebrauchsanweisung
Mode d'emploi
Instrucciones de manejo

Enraf-Nonius B.V.
Vareseweg 127
P.O. Box 12080
3004 GB Rotterdam
The Netherlands
Tel.: +31-(0)10 – 20 30 600
Fax: +31-(0)10 – 20 30 692
info@enraf-nonius.nl
www.enraf-nonius.com

Part number: 1400741-42
June 4, 2013

Parallel Bars

Bedieningshandleiding
Operating Instructions
Gebrauchsanweisung
Mode d'emploi
Instrucciones de manejo

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3446257 Loopbrug

1 Foreword

We very much appreciate your purchase of the parallel bars. The parallel bars offers the possibility to practice in a very flexible way medical training therapy.

The package contains the parallel bars and the operation manual, in which you are reading now.

When developing and producing the parallel bars much consideration was given to reliability, safety, user comfort and durability. In order to be able to use your parallel bars safely, it is important that you read the instructions before use.

2 Installation

- △ Be assured that the parallel bars is standing firmly and well levelled.
 - △ The parallel bars must be installed in such a way that it's use is not limited by other objects. Keep at least 2.50 m free around the unit.
 - △ Do not use the parallel bars in "wet rooms"
-

3 Directions with regard to safety

- △ The parallel bars should not be operated or used by more person at ones.
- △ Never re-set the bar height during exercising.
- △ Be assured that the bars are firmly tightened before the parallel bars will be used.
- △ Keep spectators away from the unit when it is in use.(at least 1 metre).
- △ At all times keep away unsupervised children from the equipment.
- △ When the parallel bars are not used in the correct way, injury may occur. Supervision by a knowledgeable paramedic is therefore obligatory.

Product Liability

A law on Product Liability has become effective in many countries. This Product Liability law implies, amongst other things, that once a period of 10 years has elapsed after a product has been brought into circulation, the manufacturer can no longer be held responsible for possible shortcomings of the product.

To the maximum extent permitted by applicable law, in no event will Enraf-Nonius or its suppliers or resellers be liable for any indirect, special, incidental or consequential damages arising from the use of or inability to use the product, including, without limitation, damages for loss of goodwill, work and productivity, computer failure or malfunction, or any and all other commercial damages or losses, even if advised of the possibility thereof, and regardless of the legal or equitable theory (contract, tort or otherwise) upon which the claim is based. In any case, Enraf-Nonius's entire liability under any provision of this agreement shall not exceed in the aggregate the sum of the fees paid for this product and fees for support of the product received by Enraf-Nonius under a separate support agreement (if any), with the exception of death or personal injury caused by the negligence of Enraf-Nonius to the extent applicable law prohibits the limitation of damages in such cases.

The opposing party (product's user or its representative) shall disclaim Enraf-Nonius from all claims arising from third parties, whatever nature or whatever relationship to the opposing party.

4 Handling and Use

The parallel bars can be used for therapy to improve strength, mobility, endurance or co-ordination. We direct you to the therapy guides and reference books published on these matters.

- Adjust the height of the support bars of the parallel bars according to the anatomy of the patient. He/she should be able to hold the bars with slightly flexed arms.
- ⚠ Always make sure that the therapist is able to support the patient in case he might fall. To achieve this the therapist should be standing / walking behind patient. The therapist should stabilise himself with one hand on the support bar, and support the patient with his other hand. Walk non-synchronised with the patient so you always stand firm when the patient is moving. In case the patient is too heavy to be supported by one therapist, make sure that extra persons are involved to assure safe support.

5 Contra-indications

5.1 Absolute contra-indications:

- Myocardial dysfunction in an unstable phase
- HNP in the (sub) acute phase (with irradiation and neurological signs)
- Severe osteoporosis
- Acute fractures
- Severe mechanical instability
- Early post-surgical joint modification

5.2 Relative contra-indications:

- Myocardial dysfunction in a stable phase
- HNP in the sub-acute phase (with irradiation and neurological signs)
- Mild osteoporosis
- Late post-surgical joint modification
- Tumours
- Inflammations in the involved area
- Non desired articular load in the involved area (p.e. with M. Osgood-Schlatter, bursites, etc.)
- Non desired ligamentous load in the involved area (p.e. sprain with hydrops)
- Non desired soft tissue load in the involved area (p.e. with severe atrophy)
- Cardiovascular restrictions
- Pulmonary restrictions
- Psychological restriction
- Pain of such cause that movement would not relieve

6 Cleaning and maintenance

- The parallel bars can be cleaned with a damp cloth and a (non- abrasive) liquid household cleaner.
- ⚠ At regular intervals do confirm the proper operation of the bar height adjustment locks.
- In order to obtain spare parts, please contact your supplier.

6.1 Environment Information

Your parallel bars contains materials which can be recycled and/or are noxious for the environment. At the end of the duration of life, specialised concerns can take apart the parallel bars and sort out the noxious materials and materials for recycling. By doing so you contribute to a better environment.

Please ensure that you are well informed of the local rules and regulations regarding to the removal of equipment and accessories.

7 Conclusion

We hope that you will use the parallel bars for a long time with much satisfaction and we take the liberty of drawing your attention to the many other high quality products in our programme. If you should have any questions or wish to receive our catalogue, please contact our representative.

8 Technical details

Dimensions (HxWxD): 66-108 x 90 x 400 cm.

Weight: 67 kg.

Height adjustment handrails: from 66 to 108 cm.

Maximal patient load: 135 kg.

Environment conditions for transport and storage

Environment temperature: -10° till +50° C

Relative humidity: 10 till 95 %

Atmospheric pressure : 500 till 1060 hPa

Environments conditions for normal use

Environment temperature: 10° till 40° C

Relative humidity: 10 till 90 %

Atmospheric pressure : 500 till 1060 hPa

Medical classification: Class I

This product complies with all essential requirements of the Council Directive 93/42/EEC of the European communities, concerning medical devices.

Technical modifications reserved.

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